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Dermabrasion

Warning

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

Recycling

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

EC-Declaration of compliance

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2006/95/EC
2. Conditions set out in the Electromagnetic Compatibility Directive: 2004/108/EC

Welcome!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

weelko
the wellnes
company

Barcelona
Pol. Ind. La Valldan
Cami de Garreta, 86
08600 Berga
Barcelona – Spain
T. +34 902 041 563
E. info@weelko.com
W. www.weelko.com

Front panel

Instructions of use


1. Connect the power supply via the rear panel.
2. Turn on the "POWER" button.
3. Correctly connect one side of the pipe to the rear panel as indicated in the sketch.
4. Connect the opposite side of the pipe to the vacuum handle.
5. Place the correct diamond head in the handle. There are nine types of diamond heads.
8. You can adjust the suction effect by turning the regulator.
9. You can adjust the suction effect by covering or uncovering the hole in the side of the handle with your finger.
10. Once you have finished, first turn off the unit by pressing the "POWER" button.

Ø 22 MM	
D100	Body. Suitable for acne, knees, elbows and areas with stretch marks
D180	
Ø 19 MM	
D180	Face. Suitable for large spots, pores, aging sun exposure
D220	
Ø 15 MM	
D100	
D200	Scars, fine lines and various imperfections
D250	
Ø 8 Y 6 MM	
D200	Eyes and both sides of the nose. Wrinkles, acne, pimples and blackheads
D250	

Cautions

- Do not leave the handle on the same part of skin for too long since it can cause injuries.
- Avoid using it on irritated skin.
- The following skin areas are unsuitable for these treatments: skin tumors, moles, birthmarks, angiomas, lips, eyes, herpes and warts.
- Do not operate the suction pump for prolonged periods without cooling time between treatments to prevent pump overheating and damage to the machine. Damage caused by overheating of the pump are not covered by the manufacturer's warranty.

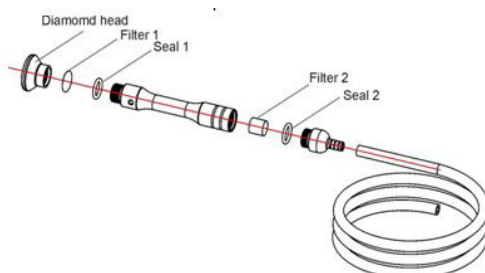6. Check that the foam filter is in its proper position in the handle.
7. Check that the metal filter is in its proper position in the handle.


FAQ

If the suction effect is reduced:

- Replace the metal and foam filters.

If the device produces an abnormal sound:

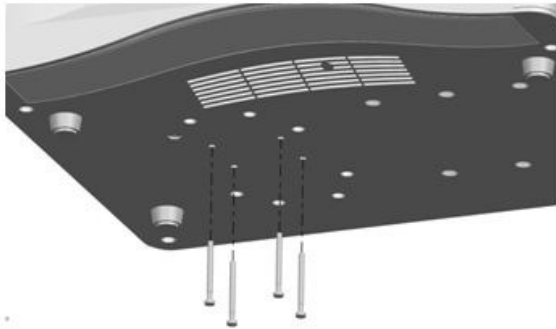- Remove the 4 screws indicated below. These screws are intended to fasten the pump during transport of the equipment. It might disrupt the correct pump vibration during operation and for that reason will increase the noise of the device.

MAINTENANCE

- The foam filter in the handle needs to be changed after every 5 treatments. Dismantle the handle and replace the filter. Then reassemble the handle.
- The metal filter in the handle needs to be changed after every 5 treatments. Dismantle the handle and replace the filter. Then reassemble the handle.
- Clean and sterilize the diamond heads after every treatment.

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CE Conformity Declaration

This product complies with the following requirements:

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Welcome!

Thank you very much for making this purchase. Please read the instructions carefully before using the device. You can also ask for advice from a professional to ensure you use the device correctly. We do not accept any liability in the event of an accident caused by incorrect use.

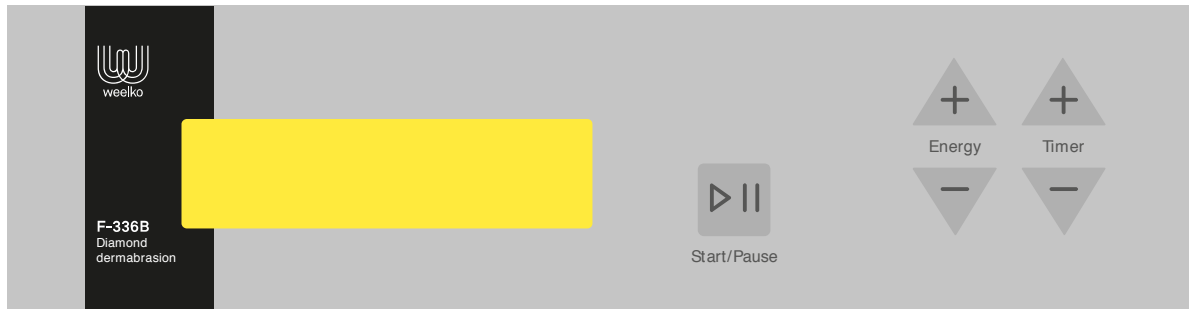
The company reserves the right to modify the details of the instruments without notice. If you detect an error in these instructions, please do not hesitate to let us know so we can provide a solution.

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